

## Instruction For Use

### **Absorbent Paper Points**

#### **Sterilized endodontic points**

Used to remove moisture from the root canal or to introduce medicaments.

#### **Composition**

Paper 100%

#### **Application**

1. Check if there is any abnormality in the appearance of the product such as foreign substances or cracks.
2. Check if the product is within the expiration date.
3. After forming the root canal, select the size that fits the file and dry the root canal, and use it with medication after.
4. This product is a gamma ray sterilized disposable medical device.

#### **Intended Purpose**

1. Intended Use: Used to remove moisture from the root canal or to introduce medicaments.
2. Indication: Used to remove moisture from the root canal or to introduce medicaments.
3. Contraindication: No contraindication has been reported that can cause health problems.
4. Patient Population: All age group
5. Intended user: Dentist
6. Side Effects/Adverse Effects: There are no side effects or adverse effects reported.
7. Potential risk if re-used: Re-use of used Paper point may cause inflammation.
8. Can be used with medicaments and tools and it shall be used under the control of the dentist according to the clinical indication.
9. Single Use Device

#### **Precaution**

1. Do not use other than dental specialist.
2. Use disinfected tools.
3. This product is a gamma-ray sterilized disposable medical device.


#### **Storage**

Store at room temperature.

**Disposal** : Dispose of contents/container according to the related regulations of the country.

**Shelf Life** : 4 years from the date of manufacture

#### **Symbols**

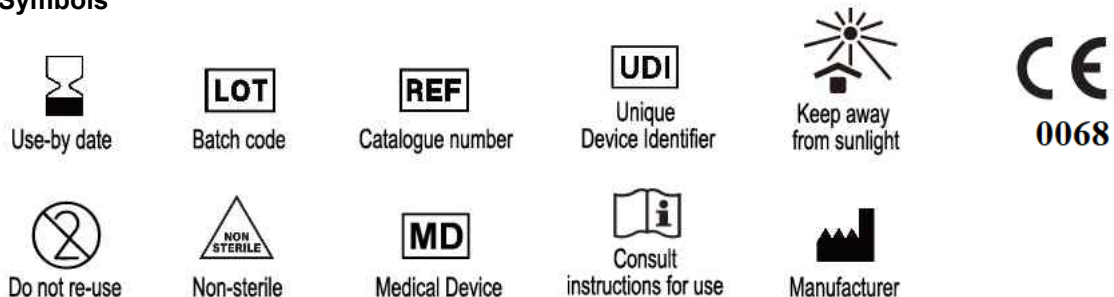

■ In case that any serious incident that has occurred in relation to the device, please report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.